




## INSTRUCTIONS FOR USE > INSTRUKTION FOR BRUG


### TWO-HANDED APPLICATION > TO-HÅNDSBETJENING

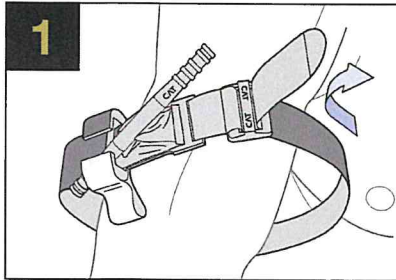


Route the band around the limb, pass the tip through the slit of the buckle, and position the **CAT** 2-3" above the bleeding site directly to the skin.\*  
 > Placerer Velcro båndet omkring ekstremiteten og før Velcro båndet gennem friktionsspændet. Placer **CAT** 5-7, 5cm. over det blødende sted, direkte på huden. Stram **CAT** til blødningen stopper.\*

OR

### ONE-HANDED APPLICATION > EN-HÅNDSBETJENING

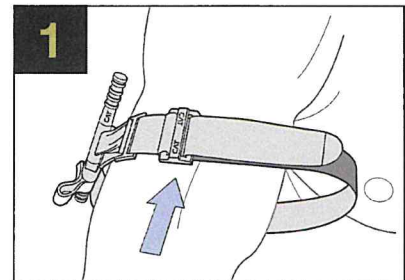


Insert the injured limb through the loop in the band and position the **CAT** 2-3" above the bleeding site directly to the skin.\*  
 > Indsæt den skadede ekstremitet gennem loopet i Velcro båndet. Placer **CAT** 5-7, 5cm. over det blødende sted, direkte på huden.\*

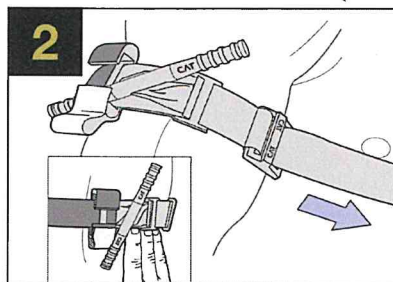


Pull band tightly and fasten it back on itself all the way around the limb, but not over the rod clips. Band should be tight enough that tips of three (3) fingers cannot be slid between the band and the limb. If the tips of three (3) fingers slide under band, retighten and re-secure.

> Stram Velcro båndet og sikre det mod sig selv. Fastgør Velcro båndet omkring ekstremiteten, dog ikke forbi drejepinden og plastik låsen. Velcro båndet skal være så stram at man ikke kan få tre (3) finger spidser mellem velcro båndet og ekstremiteten. Kan der placeres tre (3) finger spidser mellem båndet og ekstremiteten, strammes den yderligere.

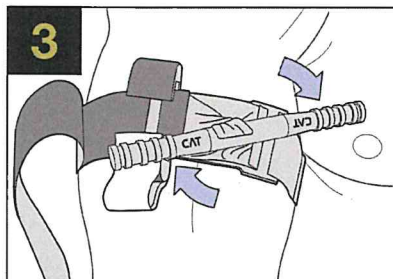


Twist the rod until bleeding has stopped.  
 > Drejepinden drejes til lys rød blødning og distal puls er forsvundet

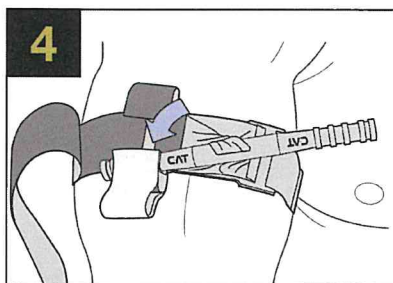

Secure the rod inside a clip to lock it in place. Check for bleeding and distal pulse. If bleeding is not controlled, or distal pulse is present, consider additional tightening or applying a second **CAT** above and side-by-side to the first. Reassess.

> Lås drejepinden i plastik låsen. revurder blødning og distal puls. Hvis blødningen stadig ikke er stoppet, stram yderligere eller anlæg yderligere en **CAT** umiddelbart over den allerede anlagte **CAT**. Revurder!

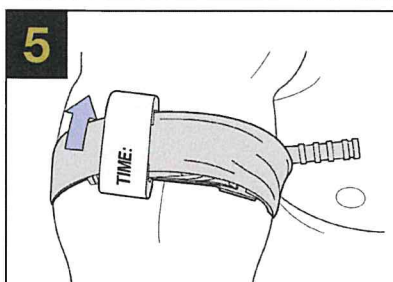


Route the band between the clips and over the rod. Secure rod and band with TIME strap. Record time of application.

> Før Velcro båndet gennem plastik låsen og over drejepinden. Drejepinden og Velcro bånd sikres i plastik låsen med velcro bånd (TIME). HUSK AT PÅFØRE TID FOR ANLÆGGELSE!

\*If you cannot be sure or cannot take the additional time to examine where the bleeding is coming from based on the situation, the **CAT** can be effectively applied over clothing as high on the arm or leg as possible. The **CAT** must NOT be applied over solid objects within the clothing. As soon as the situation permits, the injured limb should be evaluated and the **CAT** re-positioned 2"-3" above the injury directly to the skin. > \* Hvis man ikke med sikkerhed kan identificere blødningen eller hvis situationen ikke tillader det, kan **CAT** påføres uden på beklædning. Placeres så højt på ARM eller BEN som muligt. Vær opmærksom på at **CAT** ikke placeres ovenpå indhold i lommer etc. Så hurtigt som muligt bør skaden vurderes og **CAT** re-placeres 5-7, 5 cm. Over det skadede sted, direkte på huden.




**STORING THE CAT IN THE QUICK LAUNCH CONFIGURATION > OPBEVARING AF CAT I "QUICK LAUNCH" KONFIGURATION**

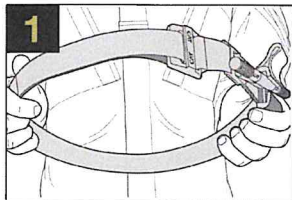


Pass the tip through the slit in the buckle. Pull 8" of band through, fold it back and adhere the band to itself.

> Før ca. 20 cm af Velcro båndet gennem revnen i friktionsspændet og lad Velcro båndet klæbe mod sig selv.

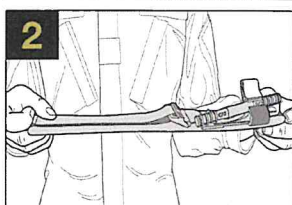

Flatten the loop formed by the band. Place the buckle in the middle of the flattened band.

> CAT foldes fladt således at friktionsspændet befinder sig i midten.

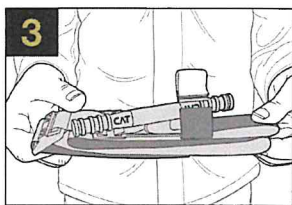

Fold the CAT in half placing the buckle at one end. Make sure the TIME strap is open and in ready position; as shown in the illustration.

> Fold CAT fladt således at friktionsspændet befinder sig for enden. Vær OBS på at TIME snippen er åben og i klar position; som vist på illustrationen.

The CAT is now ready to be placed in your medical kit.  
> CAT er nu klar til opbevaring i holder, IFAK el. lign.

**STANDARD USE GUIDELINES**

Intended Use of the Combat Application Tourniquet® (CAT) is to occlude blood flow of an extremity in the event of a traumatic wound with significant hemorrhage. Clinical benefit of the CAT is the control of a life-threatening bleed which results in less blood loss. Intended user of the CAT should only be used as directed by user's military service component guidelines, EMS authority, or under the supervision of a physician, and in strict compliance with these Instructions for Use. Target population for the CAT is anyone with major bleeding to the upper or lower extremities, which is life threatening and cannot be stopped otherwise. The CAT has a Maximum Operating Circumference of 35in (88.9cm). The use of any tourniquet for longer than 2 hours may lead to permanent neurological or muscular damage. If you cannot be sure or cannot take the additional time to examine where the bleeding is coming from based on the situation, the CAT can be effectively applied over clothing as high on the arm or leg as possible. The CAT must NOT be applied over solid objects within the clothing. As soon as the situation permits, the injured limb should be evaluated and the CAT re-positioned 2"-3" above the injury directly to the skin. The CAT should be stored in a cool, dry place away from direct sunlight. Re-use of this single use device may introduce risk of infection. Dispose of used CAT in accordance with local regulations for biomedical waste. Please report all serious incidents in relation to the CAT to the manufacturer and appropriate regulatory authority.

**> RETNINGSLINJER FOR STANDARDBRUG**

Tilsigtet anvendelse af Combat Application Tourniquet® (CAT) er at okkludere blodgennemstrømning i en ekstremitet i tilfælde af et traumatisk sår med signifikant blødning. Klinisk fordel ved CAT er kontrol med en livstruende blødning, hvilket resulterer i mindre blodtab. Tilsigtet bruger af CAT bør kun bruges som anvist af brugerens retningslinjer for militærtjeneste, EMS-myndighed eller under tilsyn af en læge og i nøje overensstemmelse med denne brugsvejledning. Målpopulation for CAT er enhver med alvorlig blødning i øvre eller nedre ekstremiteter, hvilket er livstruende og ellers ikke kan stoppes. CAT har en maksimal driftsomkreds på 88,9cm (35in). Brug af en hvilken som helst turnet i mere end 2 timer kan føre til permanent neurologisk eller muskelskade. Hvis du ikke kan være sikker eller ikke kan tage den ekstra tid til at undersøge, hvor blødningen kommer fra, baseret på situationen, kan CAT effektivt påføres over tøj så højt på armen eller benet som muligt. CAT MÅ IKKE påføres på faste genstande i tøj. Så snart situationen tillader det, skal det skadede lem evalueres og CAT placeres 2"-3" over skaden direkte på huden. CAT skal opbevares et køligt, tørt sted væk fra direkte sollys. Genbrug af denne enhed til engangsbrug kan medføre risiko for infektion. Bortskaf brugt CAT i overensstemmelse med lokale regler for biomedicinsk affald. Rapportér alle alvorlige hændelser i forbindelse med CAT til producenten og den relevante regulerende myndighed.



TOURNIQUET > TOURNIKET

BARCODE HERE



INST-008 REV04 2021-06-22



Distribution in Sweden, Norway, Denmark & Finland by:  
AP Services +45 96907090  
info@ap-services.dk www.ap-services.dk



TACTICAL BLACK (Taktisk, SORT)  
NSN 6515-01-521-7976



TRAINER BLUE (Træner, BLÅ)  
NSN 6910-01-560-2972



EMS ORANGE

Lot Number > Partinummer		Consult Instructions for Use -> Se brugsanvisningen	
Manufacturer > Fabrikant	EC REP Authorized representative in the European Union > Autoriseret repræsentant i Den Europæiske Union	Do not re-use > Må ikke genbruges	
Keep dry > Hold det tørt	Keep away from sunlight > Holdes væk fra sollys	Medical device > Medicinsk udstyr	UDI Unique Device Identifier > Unik enhedsidentifikator
Combat Application Tourniquet® and CAT are trademarks registered in the United States and certain other countries.			
U.S. Patent Nos. 7,842,067; 7,892,253; 8,888,807; 10,016,203; Canada Patent No. 2,569,550; EU Patent Nos. 1,753,344; 2,425,786; Israel Patent No. 179769; Hong Kong Patent No. 1,163,479; Other Patents Pending		Manufactured by C-A-T Resources, LLC 483 Lakeshore Pkwy, Rock Hill, SC 29730 United States of America 803.325.9300	