Model 125 KED®




Model 125

A	33" (830 mm)
B	35" (900 mm)
C	32" (800 mm)
D	11" (270 mm)
E	1" (30 mm)
F	6" (140 mm)

	7 lb, 11 oz (3 kg)
--	--------------------

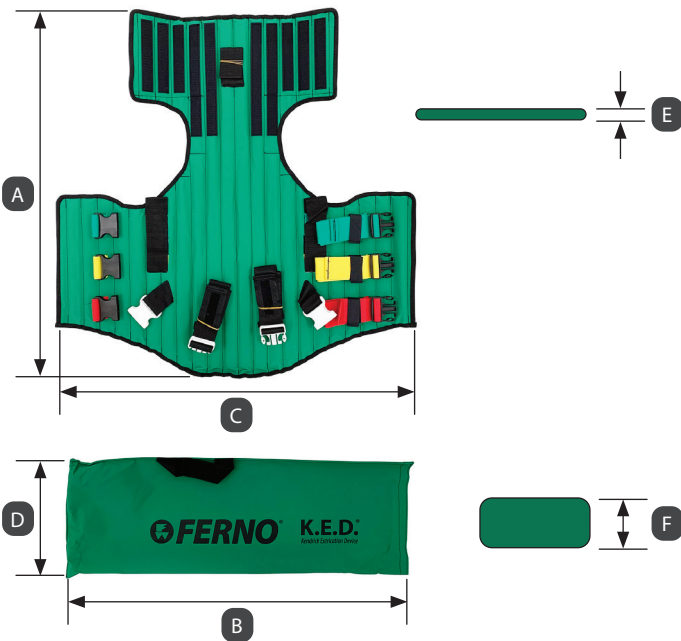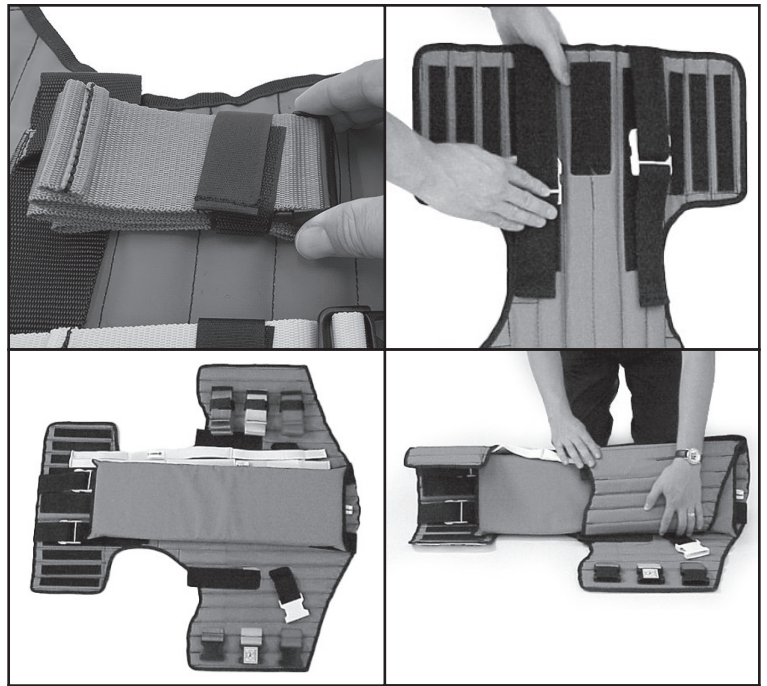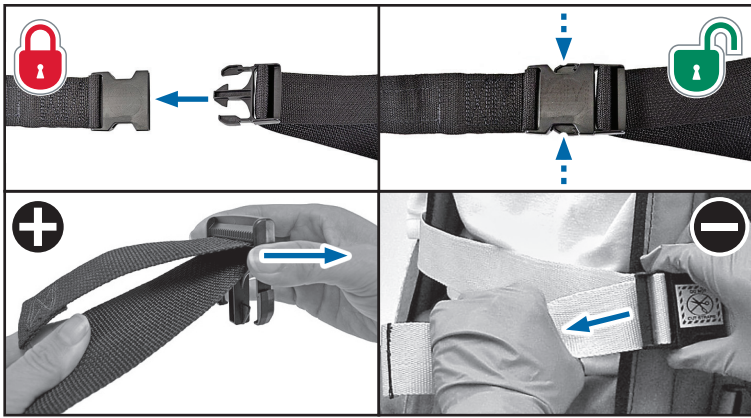	500 lb (227 kg / 35.7 st)
--	---------------------------

Ferno-Washington, Inc.
70 Weil Way
Wilmington, OH 45177-9371, U.S.A.
1.937.382.1451

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Ferno (UK) Ltd.
Ferno House, Stubs Beck Lane, Cleckheaton,
West Yorkshire, BD19 4TZ, UK
+44 (0) 1274 851 999

Description

The Ferno® Model 125 Kendrick Extrication Device (KED®) is an emergency patient-handling device designed to aid in the immobilization and short transfer movement of patients with suspected spinal/cervical injuries. It is radiographically translucent.

Features include color-coded, sewn-in securing straps; wraparound head and torso flaps, head/chin straps; an Adjusta-Pad™ for padding; integrated carrying handles; and a carrying case.

General Guidelines for Use

The KED is a rigid device with a variety of strapping configurations and is adaptable to many emergency medical situations. Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices during use of the product. A minimum of two trained operators is required. Operators must:

- Read and understand this manual.
- Train and practice on proper use of the product following standard emergency patient-handling procedures. Keep training records.
- Inspect the product and confirm it is in good condition.
- Communicate with one another and use coordinated movements to maintain control of the product.
- Stay with the patient at all times.
- Use additional trained help as needed to assist moving and carrying the KED with a patient.
- Maintain manual stabilization of the patient's head and neck until the patient is properly secured.
- Application: Position KED behind patient with torso flaps wrapped around patient and firmly against armpits. Sit patient back fully against KED as condition allows.
- Secure straps in following order: Middle torso (yellow), bottom torso (red), 2 leg straps (if applicable), 2 head straps, and top torso (blue).
- Patient's medical condition, body structure, and head shape determine the need for padding. Follow your local protocols for the use of padding.
- After transferring the KED with patient to a spine board or ambulance cot, secure the patient with additional restraints (not supplied) and loosen the KED's top torso strap (blue).

Disinfecting and Cleaning

Open the KED and separate the loose pieces before cleaning and disinfecting.

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

Hand clean with warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

Ferno Customer Relations

For ordering assistance or general information, contact your Ferno distributor or Ferno Customer Relations.

Information www.ferno.com
 Authorized Service and Support ferno.com/service-and-support

U.S., Canada 1.877.733.0911
 International +1.937.382.1451
 Fax 1.888.388.1349

Complete English Users' Manual available at: www.ferno.com.

PATENTS
ferno.com/patents

WARNING

Professional use by trained operators only. Use the product only for its intended purpose.

Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.

An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.

Improper parts, modifications, or items not designed for the product can cause injury. Use only Ferno-approved parts and service. Attach only approved items. Do not modify the product.

Symbol	Definition	Symbol	Definition
	Follow training protocol		Use proper restraints
	Disinfecting		Load capacity
	Cleaning		Product weight

NOTICE

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not use abrasive materials to clean the product.

Do not return the product to its carrying case until thoroughly dry.

Inspection Checklist

- Are all components present, in good condition, and free of excessive wear?
- Is all stitching secure?
- Are vertical battens inside the KED solid and unbroken?
- Are all buckles free of visible damage and do they operate properly?

If inspection shows damage or excessive wear, remove the product from service.

Part	Description	Qty
081-9928	Head/Chin Strap	2

Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911 or via email at tscoordinator@ferno.com. If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

Disposal

Prior to disposal, clean and disinfect the product according to instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste.

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