

Models 430, 430P, 439 Restraints

430

430P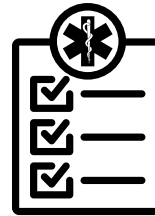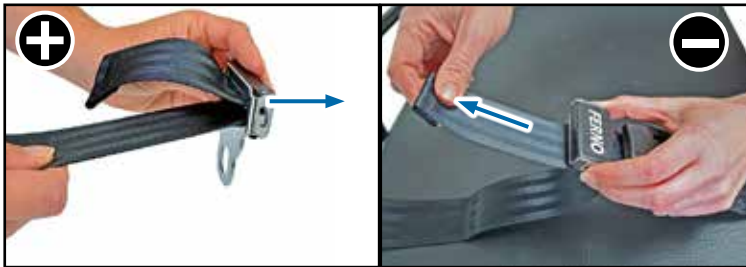

Pieces Stücke Pièces Pezzi Piezas			
2	5' (1524 mm)	031-3892	031-3891
2	7' (2134 mm)	031-3911	031-3912
2	9' (2743 mm)	031-3305	031-4013
1	7' (2134 mm)	031-3910	031-3887
1	9' (2743 mm)	031-3899	031-3889

Pieces Stücke Pièces Pezzi Piezas			
2	5' (1524 mm)	031-3797	031-3917
2	7' (2134 mm)	031-3801	031-3919
1	7' (2134 mm)	031-3894	031-4011

Pieces Stücke Pièces Pezzi Piezas			
1	7' (2134 mm)	031-3083	031-2660

Ferno-Washington, Inc.
70 Weil Way
Wilmington, OH 45177-9371, U.S.A.
1.937.382.1451

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Ferno (UK) Ltd.
Ferno House, Stubs Beck Lane, Cleckheaton,
West Yorkshire, BD19 4TZ, UK
+44 (0) 1274 851 999

Description

Ferno® restraints provide a variety of strapping options to help secure a patient on emergency-medical equipment. Select the restraint that matches the intended use.

- Model 430 Primary-Use Restraints have metal buckles and tangs, and are suitable for securing a patient on compatible equipment during transport in a motor vehicle.
- Model 430P Secondary-Use Restraints have plastic buckles and tangs, and are not suitable for use in a motor vehicle. They are for temporary use to secure a patient on a backboard, hand-carried stretcher, scoop stretcher, stair chair, or extrication device. Transfer the patient onto an ambulance cot or other device with primary restraints before transport in a motor vehicle.
- Model 439 is a set of three Model 430 metal-buckle restraints.

General Guidelines for Use

Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices during use of the product. Operators must:

- Read and understand this manual.
- Train and practice on proper use of the product following standard emergency patient-handling procedures. Keep training records.
- Inspect the product and confirm it is in good condition.
- For placement on a patient, follow local protocols. Adjust restraints to safely secure the patient to the equipment without causing discomfort or impairing circulation. Stay with the patient at all times.
- Secondary-use (plastic-buckle) restraints are not for use in a motor vehicle.
- Keep restraints fastened when not in use to prevent interference with equipment operation.
- To attach two-piece restraints, wrap the loop around the attachment point and insert the buckle or tang through the loop, then pull tight (page 1).
- To attach one-piece restraints, thread restraint through attachment points and around product. Follow local protocols.

Load Capacity

Refer to the equipment users' manual for the equipment load limit. Do not exceed equipment load capacity.

Disinfecting and Cleaning

Remove restraints from the product before cleaning and disinfecting.

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

430: Do not immerse metal buckles in liquid. Immerse webbing in solution of warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

430P: Hand clean with warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

Ferno Customer Relations

For ordering assistance or general information, contact your Ferno distributor or Ferno Customer Relations.

Information	www.ferno.com
Authorized Service and Support	ferno.com/service-and-support
U.S., Canada	1.877.733.0911
International	+1.937.382.1451
Fax	1.888.388.1349

Complete English Users' Manual available at: www.ferno.com.

WARNING

Professional use by trained operators only. Use the product only for its intended purpose.

Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.

An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.

Improper parts, modifications, or items not designed for the product can cause injury. Use only Ferno-approved parts and service. Attach only approved items. Do not modify the product.

Symbol	Definition	Symbol	Definition
	Follow training protocol		Model 430 metal-buckle restraint
	Disinfecting		Model 430P plastic-buckle restraint
	Cleaning		Lock
	Lengthen		Unlock
	Shorten		Load capacity

NOTICE

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not use abrasive materials to clean the product.

Do not immerse metal buckles in liquids or place in washing-machine.

Inspection Checklist

- Are all components present?
- Is webbing in good condition with no cuts or frayed edges?
- Are buckles free of visible damage and do they operate properly?
- Can strap length be adjusted smoothly and properly?

If inspection shows damage or excessive wear, remove the product from service and replace.

Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911 or via email at tscordinator@ferno.com. If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

Disposal

Prior to disposal, clean and disinfect the product according to instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste.

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