

Additional Safety Instruction KangooFix -Usage with 4-point "X-shaped" stretcher belt system.

Product Name:	KangooFix Ready2go Set
Germa Article No:	15271000819/ 15271000820
Description:	KangooFix is a harness system for safe ambulance transports of a new born child. The system shall be assembled on stretchers belt system.

Safety Instruction

Triglidge belt adjusters, Germa article number 12078600050, must be assembled on stretchers 4-point shoulder "X-shaped" safety belts before using KangooFix.


The Triglidge belt adjusters shall be assembled at upper chest height on the left and right safety belts, according to pictures below.

Note that in following pictures, the Triglidge belt adjuster is mounted too close to the buckle (the buckle is used as a guidance to clarify the assembly procedure).


1. Enter belt through the lower hole of the belt adjuster, make sure the arched side of belt adjuster is directed from the male buckle;



2. Enter belt through the upper hole of the belt adjuster;

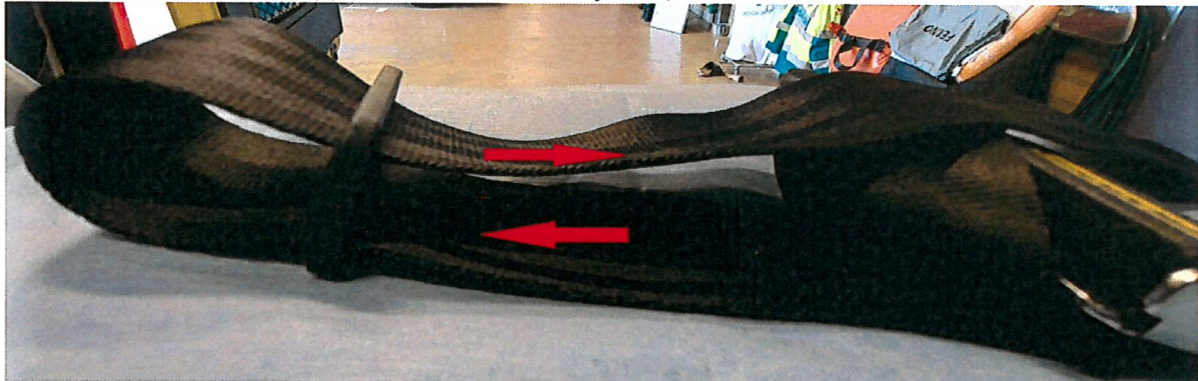

3. Loop the belt and turn back on arched side of belt adjuster, and through the lower hole of the belt adjuster;

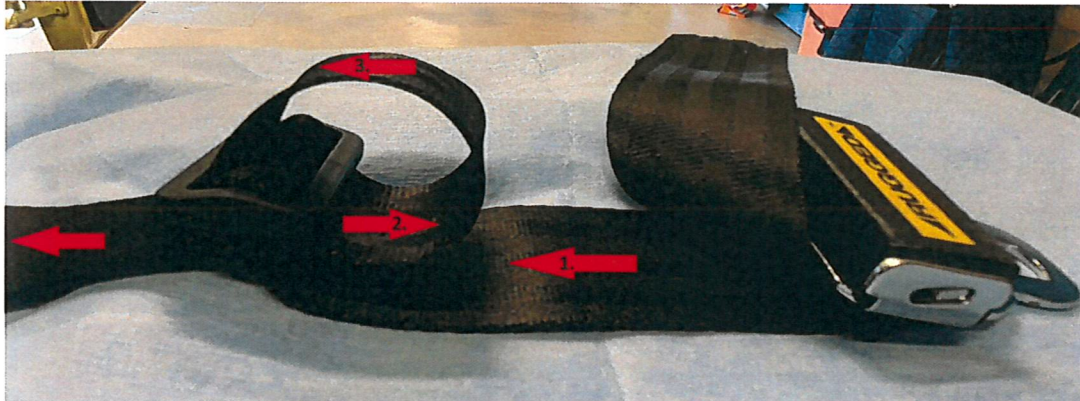

After the Triglide belt adjusters are assembled, the KangooFix harness can be connected to stretchers belt system, following KangooFix Instructions for use, Germa article number 1244300000.

1. Connect the waist straps of the harness behind each female buckle of the X-shaped belt by making loops around them, then tighten to remove slacks.
2. Connect the upper leg waist strap of the harness behind the female buckle of the upper leg belt by making a loop around it, then tighten to remove slacks.
3. Connect the shoulder straps of the harness over the belt adjusters of the two belts by making loops around them, then tighten neatly to remove slacks.



We disclaim all liability and any damages for use in another combination.

Signed in Kristianstad, Sweden on behalf of AB Germa

2022-04-26



Lina Reffers,
Quality and Technical Manager

Postal address

AB GERMA
Box 2127
SE-291 02 KRISTIANSTAD
SWEDEN

Goods address

Industrigatan 54-56
SE-291 36 Kristianstad

Phone

+46 (0)44-123030

Fax

+46 (0)44-10 31 79

E-mail/Web page

info@germa.se
www.germa.se

VAT No. Bank account

SE556086941301

Bank account

481-4125