DIRECTIONS FOR USE

Ferno Heavy Duty Transfer Sheet

English: Directions for use. **Ferno Stretcher/Transfer Mattress**

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English: These directions for use may be updated without further notice.
Copies of the current version are available from the manufacturer.

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1. Warning and caution statements

CAUTION

- Avoid handling the mattress next to sharp objects.
- Avoid handling the mattress next to open fire.
- On a current basis check that the mattress has no cracks.

Recommendation: Usable life of 3 years

2. Intended use

The **Ferno Heavy Duty Transfer Sheet** is intended to be used as a transfer sheet during patient handling and transportation. It is also a device intended to facilitate the transfer of the patient from one stretcher to another. (EN 1865).

The transfer sheet can be carried by minimum 2 persons for a shorter distance, and a longer distance two or more persons use additional carrying belts. You can pull the mattress on the ground by the handles or by connecting a carrying belt or rope.

The mattress consists of an fully welded, material that is and is fitted with 20 handles on two or four sides for carrying or sliding / pulling.

3. Specifications

The Ferno Heavy Duty Transfer Sheet is tested in accordance with EN 1865.

Contains an anti-fungal agent to control microbial deterioration. The products do not contain any nano materials. Biocompatible material. This product is in conformity with Council Directive 93/42/EEC concerning Medical Devices and EN 1865.

Temperature range: -30°C to +70°C

Parts/Materials

Outer shell: PVC coated fabric (flame retardant)

Reverse material: Polyester glassfiber material PVC

Handle: Polyester roop

Dimensions	Length (cm)	Width (cm)	Height (cm)	Weight (kg)	N °of handles
Ferno Stretcher/Safe Transfer Mattress 12	205	143	1,3	2.4 / 2.6	20

4. Description

The Ferno Heavy Duty Transfer Sheet is equipped with 6 + 4 + 6 + 4 handles at each side

The mattress consists of an outer cover PVC coated polyester weaved canvass with reverse of polyester glassfiber PVC material

The sheet is equipped with polyester fiber back for better sliding on ground and between stretchers.

5. Operating instructions

1. Put the sheet on the floor/bed/ground
3. Place the injured patient in the medically desired position on the Ferno Heavy Duty sheet.
4. Check that the unit is stable and comfortable for the patient.
5. The injured patient is ready to be slided/carried and transported.
6. When patient transfer and need of lifting the patient up, make sure that the team of rescuers has the required capacity to lift a heavy duty patient in ergonomic procedure.
7. The Ferno Heavy Duty Transfer Sheet can easily be slided over to another stretcher or trolley at the hospital or at change of transportation.
8. Make sure that you can secure the patient on top of a stretcher using harness system from the stretcher.

6. Cleaning and Disinfecting

At a regular basis, the Stretcher/Transfer mattress should be wiped with a moist cloth and dried, if dirty, the mattress may be washed with soapy water and rinsed with clean water.

Allow the mattress to dry before storage.

If the transfer mattress needs disinfection first clean the mattress carefully and let it dry. Pour a solution containing 70 % alcohol or the disinfectant called LYSETOL AF on a clean cloth. Smear the transfer mattress with the cloth.

Mattress can be cleaned with most common disfectants

7. Repair of Ferno Heavy Duty Sheet

If damage occurs we do not recommend repair for safety reasons, this may decrease the carry capacity of the product.

8. Storage

The Ferno Heavy Duty Transfer Sheet is often stored in a ambulance storage room or cand be stored on the stretcher.

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Report no. :

(To be assigned in by
Manufacturer)

Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.

For claims please specify :

Purchase Date

Description of product

Description of Incident/Problem: 1) What, 2) How?

When? In connection with: 1) transport, 2) unpacking, 3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly, 10) storage, 11) other:

Info on the product, prior to problem,

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

Used cleaning and disinfecting agents/Sterilizing method:

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

Which **actions** have been: a) taken, b) planned by the reporter or user at the site of incident?

1) examination of the product on site, 2) replacement of the product, 3) other:

Suggested actions by the distributor/complainant

Returning of samples/faulty product(s),

- 1) Already shipped/Will be shipped to _____,
- 2) Pending request from manufacturer,
- 3) Product/parts not available:

Date:

We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.

Your contact person in this matter is: _____

Name:

Direct Tel.

Direct Fax.

Init/Date: