DIRECTIONS FOR USE

STRETCHER/TRANSFER MATTRESS

English: Directions for use. **Saver Stretcher/Transfer Mattress**

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1. Warning and caution statements

CAUTION

- Avoid handling the mattress next to sharp objects.
- Avoid handling the mattress next to open fire.
- On a current basis check that the mattress has no cracks.
- Own modifications on products not allowed – Your medical specialist to be responsible for any modification.
- When decided to take the product out of service and destroy – make sure to follow your local instructions or contact us for additional information.

Recommendation: Usable life of 3 years

2. Intended use

The **Heavy Duty Transfer Mattress** is intended to be used for handling of heavy duty patient a comfortable mattress on to transfer the patient from ground or floor and down stair cases and on a heavy duty stretcher during transportation. It is also a device intended to facilitate the transfer of the patient from one stretcher to another or hospital bed. (EN 1865).

The mattress can be slid on ground/floor or carried by min. two or more persons for a shorter distance, and a longer distance if two or more persons use inbuilt slide belts and additional carrying belts. You can pull the mattress on the ground by the straps and handles or by connecting a carrying belt or rope.

The mattress consists of an outer shell, fully welded, comfort part consist of a foam inlay and patient safety harness and cover part to secure the patient. The mattress is fitted with handles on two sides for carrying or pulling plus sliding straps with handles.

3. Specifications

The Saver Stretcher/Transfer Mattress is tested in accordance with EN 1021-1

Temperature range: -30°C to +70°C

Parts/Materials

Outer shell: PVC coated fabric (flame retardant)

Filling material: Foam of polyester

Reverse: reinforced with fiberglass material

Safety belt: Polyester and Fix Lock

| Dimensions | Length (cm) | Width (cm) | Height (cm) | Weight (kg) | N ^o of handles |
|-------------------------------------|-------------|------------|-------------|-------------|---------------------------|
| Heavy Duty Transfer Mattress | 205 | 143 | 8 | 8 | 16 |

4. Description

The Heavy Duty Transfer Mattress is equipped with 6 handles at each side and with 2 straps dressed with handles at each end, head and foot. Two safety belt to secure the patient onboard the mattress.

The mattress consists of an outer cover of hard-wearing PVC coated polyester weaved canvas with a foam filling.

5. Operating instructions

1. Place the injured patient in the medically desired position on the Heavy Duty Transfer Mattress.
2. Make sure to secure the patient onboard the mattress using 2 safety belts on the mattress and cover the patient with the cover part above the patient. Make sure the patient is secured and comfortable.
- 3 Start carefully to slide by pulling the mattress on the ground/floor stretcher, handles on the side for manouvering and if needed to lift or carry. Can be slided down staricases – make sure to have manpower to cover for the weiht of the patient in this process.
3. For ambulance transport lift up and place the mattress and patent on the stretcher. Make use of the safety belts of the stretcher to secure the patient according to specification for the stretcher.
4. Check that the unit is stable and comfortable for the patient.
5. The injured patient is ready to be transported.
6. The Heavy Duty Transfer Mattress dimensions allows it to fit any most commonly sold Heavy Duty stretchers.

6. Cleaning and Disinfecting

At a regular basis, the Stretcher/Transfer mattress should be wiped with a moist cloth and dried, if dirty, the mattress may be washed with soapy water and rinsed with clean water.

Allow the mattress to dry before storage.

If the transfer mattress needs disinfection first clean the mattress carefully and let it dry. Pour a solution containing 70 % alcohol or most common disinfectants on a clean cloth. Smear the transfer mattress with the cloth.

7. Repair of Stretcher/Transfer Mattress

If damage occurs repair as follows:

- a) Find a patch in a suitable size and put glue on both the patch and the damaged spot on the transfer mattress.
- b) Wait for the glue to become dust dry (approx.2 min.) and place the patch on the damaged spot.
- c) Press firmly the whole area of the patch, so that there is a good contact and all air is pressed out under the patch.

The repair will receive full strength after 24 hours.

8. Storage

The Heavy Duty Transfer Mattress to be stored in the ambulance or inside a storage building.

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Report no. :

(To be assigned in by
Manufacturer)

Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.

For claims please specify :

Purchase Date

Description of product

Description of Incident/Problem: 1) What, 2) How?

When? In connection with: 1) transport, 2) unpacking, 3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly, 10) storage, 11) other:

Info on the product, prior to problem,

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

Used cleaning and disinfecting agents/Sterilizing method:

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

Which **actions** have been: a) taken, b) planned by the reporter or user at the site of incident?

1) examination of the product on site, 2) replacement of the product, 3) other:

Suggested actions by the distributor/complainant

Returning of samples/faulty product(s),

- 1) Already shipped/Will be shipped to _____,
- 2) Pending request from manufacturer,
- 3) Product/parts not available:

Date:

We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.

Your contact person in this matter is: _____

Name:

Direct Tel.

Direct Fax.

Init/Date: