AB GERMA®

INSTRUCTIONS FOR USE TRANSFER MATTRESS FLEX

English: Instructions for use. **Transfer Mattress Flex**

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Phone: +46 (0)44 12 30 30

E-mail: info@germa.se
www.germa.se

English: These instructions for use may be updated without further notice. Copies of the current version are available from the manufacturer.

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INSTRUCTIONS FOR USE TRANSFER MATTRESS FLEX

English Instructions for use

INSTRUCTIONS FOR USE TRANSFER MATTRESS FLEX

1. Introduction

The Transfer Mattress Flex hereafter named the Transfer mattress, is intended to facilitate the transfer of the patient from one stretcher to another. It is also intended to be used as a comfortable mattress on a stretcher during transportation.

The Transfer mattress should be operated with a minimum of 2 operators when carrying the Transfer mattress a shorter distance. Additional people may be required for heavier patients and when carrying longer distances. Additional carrying belts can also be used to aid manual handling.

The mattress consists of an outer shell of an airtight, fully welded, material that is filled with a foam inlay and is fitted with 5 handles on each side.

Maximum load is 250 kg, all seams are fully welded for the best hygiene and easy to clean.

Recommendation: service life of 3 years

2. Precautions

- >> Avoid handling the Transfer mattress next to sharp objects.
- >> Avoid handling the Transfer mattress next to open fire.
- >> On a current basis, check that the Transfer mattress has no cracks.
- >> Own modifications on products are not allowed – Your medical specialist to be responsible for any modification.
- >> When decided to take the product out of service and scrap it – make sure to follow your local instructions or contact us for additional information.

3. Description

- >> The Transfer mattress is equipped with 6 handles on each side.
- >> The Transfer mattress consists of an outer cover of hard-wearing PVC coated polyester fiber, a foam filling of polyether and a reinforcement plate of polypropylene.
- >> The back of the Transfer mattress is equipped with a non-friction glass fiber that provides a glide function for easy transfers and sliding of the mattress and patient e.g. from a bed to stretcher. In tight spaces where the stretcher does not reach, simply slide the Transfer mattress to the stretcher.

4. Operating the Transfer Mattress

INSTRUCTIONS FOR USE TRANSFER MATTRESS FLEX

1. Place the patient in the medically desired position on the Transfer Mattress.
2. Lift and carry the Transfer mattress and patient carefully on to the stretcher.
3. Close the belts of the stretcher.
4. Check that the unit is stable and comfortable for the patient.
5. The injured patient is ready to be transported.
6. The Transfer Mattress can also be used as a stretcher mattress. Its dimensions allows it to fit any commonly sold stretcher.
7. The Transfer Mattress can easily be slid over to another stretcher or trolley at the hospital or at change of transportation.

5. Cleaning and Disinfecting

> After each use the Transfer mattress should be wiped with a moist cloth and dried. If dirty, the Transfer mattress may be washed with soapy water and rinsed with clean water.

>> Allow the Transfer mattress to dry before storage.

>> If the Transfer mattress needs disinfection, first clean the mattress carefully and let it dry as mentioned above. Pour a commonly available disinfectant on a clean cloth. Clean the Transfer mattress with the cloth, dry off.

6. Maintenance and Repair

If damage occurs, repair as follows:

>> If any holes are found on the Transfer mattress, first clean the area around the damaged spot and allow it to dry.

>> Find a patch in a suitable size and put glue on both the patch and the damaged spot on the Transfer mattress.

>> Wait for the glue to become dust dry (approx. 2 min.) and place the patch on the damaged spot.

>> Press firmly on the whole area of the patch, so that there is a good contact and all air is pressed out under the patch. The repair will receive full strength after 24 hours. In case of emergency, the Transfer mattress can be used after 5 minutes.

7. Storage

>> Check after each use that the transfer mattress is ready for next use.

>> The Transfer Mattress is often positioned on top of the stretcher with a covering blanket or hung on the wall on hangers.

8. Specifications

The Transfer Mattress Flex is in conformity with Council MDR 2017/745 Class 1 concerning Medical Devices. The Transfer mattress complies with EN1865.

MR Safe

Dimensions:

Length x Width x Height:	see product information below
Weight:	see product information below
Heat resistance:	70°C
Cold resistance:	-30°C

Materials:

Part

Outer Shell
Filling Material
Plate:

Material

Polyester fibers PVC coated (flame resistant)
Polyether foam
Polypropylene, plywood

Product:

Art. no.	UDI-DI	Product name	Length x Width x Height, Weight
24271070000	07350019597460	Transfer Mattress Flex FNO	190 x 65 x 6/10 cm, 7,0 kg

Accessories:

15000000034	Easy Handle Set
15000000117	Step by Step Transfer Black II

Spare parts:

15000000122	Repair kit mattresses blue
-------------	----------------------------

INSTRUCTIONS FOR USE TRANSFER MATTRESS FLEX

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NOTICE TO USERS AND PATIENTS

In the event of a serious incident related to the use of this device, it is imperative to take the following steps:

1. Report the Incident to the Manufacturer:

Contact the manufacturer of the device promptly. Ensure that all relevant details about the incident, including the circumstances and any observed effects, are provided to facilitate investigation and resolution.

2. Notify the Competent Authority:

The competent authority in the Member State where you are established must also be informed about the incident. This ensures compliance with regulations and enables authorities to take any necessary actions to safeguard public health.

Your prompt reporting helps improve device safety and ensures appropriate measures are taken to prevent future occurrences. Thank you for your cooperation.